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### en — english — 167

These instructions for use can also be downloaded from:
**www.bauerfeind.com/downloads**

## Dear Customer,

Thank you for choosing a Bauerfeind product that meets very high medical and quality standards.

Please read through the instructions for use carefully and in full, because they contain important information. Please keep these instructions for use safe, as you may want to refer to them later. If you have any questions, please contact your physician, medical retailer or our customer service department.

### Intendd use

Orthopedic orthoses are functional orthoses for the feet and are suitable for use in treating and preventing foot complaints.

### Indications

The feets orthosis support:

- The early prevention of damage to the feet in patients with diabetes without polyneuropathy (PNP) and without peripheral artery disease (PAD)
- The treatment of symptoms caused by foot deformities

The treating physician is responsible for the precise diagnosis.

### Risks of using this product

⚠ The foot orthosis is not suitable for diabetics with PNP and / or PAD or rheumatism (rheumatoid foot deformities), as it does not provide soft enough cushioning for these types of foot complaints.

Before using / applying a foot orthosis for the risk mentioned above, you should consult a physician or specialist\*. Furthermore, combining an aid with other medical products (supports and compression stockings) may only be considered after having sought advice from your treating physician or specialist.

There is not enough space inside the shoe for the foot and the selected foot orthosis, or if the product is worn in shoes of an incorrect length and width (or has with inadequate lateral guidance in the case of open shoe type), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure, rubbing).

If the feet are positioned on the edges of the foot orthoses, blisters and calluses may form on the soles of the feet. The same symptoms may occur if excessive medial or lateral strain leads to holes or creases developing in the top covers, cushioning material or orthotic cores, or to the breakage of these. You must **stop** wearing foot orthoses if they are damaged or mis worn.

Creases, stockings or pantyhoses must be pulled flat before the shoes are put on, as creases of this kind can cause blisters and pressure points.

### Kontraindications

The foot orthosis is not suitable for diabetics with PNP and / or PAD or rheumatism (rheumatoid foot deformities), as it does not provide soft enough cushioning for these types of foot complaints.

The foot orthosis must not come into contact with damaged skin and / or chronic wounds.

### Application instructions for the wearer of the foot orthosis

Shoes suitable for wearing with foot orthoses must guarantee sufficient internal volume for both the foot and the selected foot orthosis (adequate shoe length / width, removable, suitable cushioned sole, adequate lateral guidance of the foot orthosis in the case of an open shoe type). An uncomfortable fit that constricts the foot must always be avoided.

When selecting the correct size of foot orthosis, the packaging information served merely as a guide. Just like correct fitting shoes, foot orthoses must be at least 1.0 cm longer than the foot, which the foot is in a weight-bearing, standing position. As feet may differ in length, please check both feet and, if in doubt, base your size selection on the longer foot.

Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated in your specific instructions for use.

The foot orthosis should be positioned straight in the shoe and have as much surface contact as possible with the shoe insole. Contact between the foot orthosis and the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet.

If you are unsure about anything during the fitting or regarding the fit, please contact a specialist. The foot orthosis must not be worn continuously for more than 24 hours. Please regularly check your feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any problems, please contact a specialist or your physician. Foot orthoses must be checked by a specialist after they have been worn for approximately half a year to ensure that they are still functioning properly.

### Cleaning instructions

Foot orthoses must be removed from the shoes on a daily basis in order to allow moisture to evaporate. The foot orthoses are not machine-washable, unless expressly indicated otherwise on the packaging.

All surface materials facing the foot can be wiped if required, using a mild cleanser and a sponge or cloth. After cleaning, any residual soap must be thoroughly rinsed off with water. The orthoses must be left to air dry and no external heat source should be used (heating, hair-dryer, direct sunlight).

Mild disinfectants suitable for human skin may be applied sparingly (sprayed on). However, this may lead to discoloration of top cover materials.

### Part of the body this product is used for

The foot orthoses must be worn in conjunction with insoles in suitable shoes with a sufficient internal volume for both the foot and the selected orthosis.

### Notes on reuse

The product is designed to provide care for an individual patient / individual foot orthoses wearer and is not to be reused by any other person. The manufacturer accepts no liability in the event of improper care of the product.

### Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. Please contact the person who gave you the product directly in the event of a warranty claim. Please note:

- Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function correctly can help you.
- Failure to follow the instructions may also restrict the warranty.

Discoloration, in particular fading of the top covers of the foot orthoses, may occur during their use. This is depending on a number of factors, such as moisture levels and sock / stocking materials. This is not covered by warranty.

### Disposal

You can dispose of the product in accordance with national legal requirements.

\* A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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