
THUASNE®

en	Sport tennis-elbow armband.....	4
fr	Bracelet anti-épicondylite sport	5
de	Sport-Armband gegen Epicondylitis	6
nl	Anti-epicondylitis armband sport	7
it	Bracciale antiepicondylite per uso sportivo	9
es	Brazalete antiepicondylitis para uso deportivo	10
pt	Bracelete antiepicondylite para uso desportivo	11
da	Armbind mod epicondylitis sport	12
fi	Epikondyliittikyynärvarsituki urheilukäyttöön.....	14
sv	Armband mot epikondylit sport.....	15
el	Αθλητική περιαγκωνίδα κατά της επιγονδυλίτιδας	16
cs	Pažní bandáž proti sportovní epikondylitidě	17
pl	Opaska zapobiegająca entezopatii stawu łokciowego (łokieć tenisisty) sportowa	18
lv	Sporta ortoze epikondilīta novēršanai	20
lt	Apyrankė epikondilito prevencijai sportuojant.....	21
et	Epikondüliidi tugiside sportimiseks.....	22
sl	Komolčna manšeta proti epikondilitisu šport	23
sk	Športová epikondylická páska.....	24
hu	Epicondylitis elleni karkötő sportoláshoz.....	26
bg	Лента срещу епикондилит за спорт	27
ro	Orteză de cot antiepicondilită sport.....	28
hr	Sportska narukvica protiv epikondilitisa.....	29

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

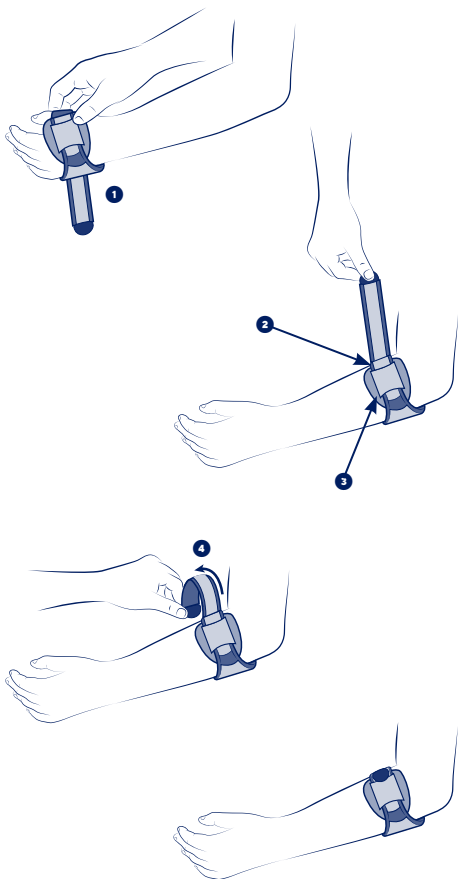en	Forearm circumference
fr	Tour de l'avant-bras
de	Umfang des Unterarms
nl	Omtrek van de voorarm
it	Circonferenza dell'avambraccio
es	Circunferencia del antebrazo
pt	Perímetro do antebraço
da	Omkreds af underarm
fi	Yläkäsivarren ympärysmitta
sv	Runt underarmen
el	Περίμετρος του αντιβραχίου
cs	Pro předloktí
pl	Opaska na przedramię
lv	Apkārt apakšdelmam
lt	Dilbio apimtis
et	Küünarvarre ümbermõõt
sl	Obseg podlakti
sk	Obvod predlaktia
hu	Alkar kerülete
bg	Обиколка на предмишницата
ro	Circumferința antebrațului
hr	Opseg podlaktice

en

SPORT TENNIS-ELBOW ARMBAND

Description/Destination

This device is an elbow brace providing epicondyle support.

This device can be used when resuming sports activities.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - polyurethane - polyester - cotton - elastane - silicone.

Properties/Mode of action

Improved proprioception.

Two pads with silicone inserts, one of which is adjustable and removable.

Adjustable pressure via self-fastening strap.

Bilateral product.

Indications

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart. In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Slip on the armband. ❶

Place the adjustment part (tightening strap + buckle) on the inside of the arm. ❷

Adjust the removable silicone insert so that it is correctly positioned over the painful area. ❸

Fasten the strap using the self-fastener. ❹

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.).

Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2051701 (2024-02)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186